

Operating Instructions

Function:

Precise and rapid measurements of pupil heights using the boxing system for all types of frame.

1. Once the frame is correctly adjusted to the patients face, take the HMS between the thumb (on A) and the forefinger (on B), opening the two mobile clips.

2. Carefully install the unit on the frame using the upper attachments and release pivoting clips on the lower part of the frame.

3. Centre the axis of the unit

● Position the white feature in the center of the pupil by using the two knobs (on C) on the sides of the unit.

● Read the tagential graduation at the bottom of the frame and note the Pupil height, taking into account the depth of the rim groove.

NB: It is advisable to take this measurement in both positions, sitting and standing to obtain a reliable average height.

